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In 100 g are contained:

2-phenoxyethanol 2,0 g,  
Alkyl (C12-16)dimethylbenzylammonium chloride 7,0 g,  
N-(3-aminopropyl)-N-dodecylpropane-1,3-diamine 5,0 g

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alcohol-free concentrate with a comprehensive spectrum of activity for disinfecting wipe-resistant surfaces of medical devices, in particular device surfaces and objects in the vicinity of patients. Remove visible soiling before disinfection; moisten surfaces completely with the solution and distribute the disinfectant solution with a hygienic, clean disposable cloth if necessary, observing the application concentrations and reaction times; keep the surface moist during the entire contact time. Use dosing aids and observe instructions; application example: 10 L of a 1% solution = 9.9 L water + 100 ml Medizid® AF+.

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2-Phenoxyethanol 2,0 g,  
Alkyl (C12-16)dimethylbenzylammoniumchlorid 7,0 g,  
N-(3-Aminopropyl)-N-dodecylpropan-1,3-diamin 5,0 g /

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**Important Note:** Non-observance of the information, especially in view of use, compatibility and durability may lead to damage to persons and / or material. Meditrade does not take any liability in case of improper use. This is a single use product – changes in material / infection risk cannot be excluded in case of multiple use. All information included correspond with the last document amendment date. Errors, literal mistakes and changes expected. In case of doubt please seek expert advice. Please find any up-to-date product information on our homepage [www.meditrade.de](http://www.meditrade.de).